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PARASITE TRANSPORT, PRESERVATION AND CONCENTRATION SYSTEM

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1/2

FEATURES

The Mini/Midi-System is a disposable plastic device for the collection, fixation, filtration and concentration by sedimentation of intestinal Parasites from fecal specimens. The collection vial is available in two sizes and with different fixatives for the morphological preservation of the parasites. The built-in filters eliminate the food debris providing a clean sediment after centrifugation. The parasites are identified by the examination of the concentrated sediment by low magnification microscopy. The concentration technique can be combined with a permanent staining to improve detection of protozoa. Trichrome staining and iron hematoxylin are recommended.

	Volume	Fixative	Presentation
Mini-System	4 ml	Total-Fix / SAF/ Ecosaf /	150 units
Midi-System	10 ml	/ FF / Empty	98u units

SAMPLE COLLECTION

1. Avoid the use of antidiarrheal medication or laxatives before taking the sample.
2. Defecate in a clean dry recipient.
3. Avoid contaminating of faeces with urine or water.
4. Pick an adequate amount of faeces and put it into the preservative tube using the cap-spoon. The quantity of sample to pick up should not exceed the capacity of the tube. **Not to open the bottom cap of the tube in any moment.**
5. Close well the tube and send it to the laboratory for processing.

KIT COMPLEMENTS (OPTIONAL)

- Sedimentation cone
- Wooden sticks
- Cone Tap
- Easy pick (Mini-System only)

WARNINGS AND PRECAUTIONS:

- Only for *in-vitro* diagnosis; professional use. NON-DOMESTIC USE.
- All samples must be treated as a potential source of infection. Ensure you follow correct laboratory guidelines at all time. It is recommended to wear gloves and **wash your hands**.
- The content of the tube is toxic, do not swallow. Avoid contact with skin and eyes. If accidental ingestion or spillage occurs, consult the safety data sheet.
- Keep away from children.
- To ensure an accurate diagnosis, it is recommended to take 3 different samples collected on alternate days.
- Do not use after expiration date.

STORAGE, STABILITY AND DISPOSAL:

The test kit must be stored at room temperature for the duration of the shelf-life indicated on the label. Dispose of using specific methods for the destruction of material with biological content. The content of the tube is toxic, consult the legislation on the disposal for toxic substances. If this is not possible, hire an authorized company for it. The container is made of 100% recyclable polypropylene.

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MATERIAL REQUIRED BUT NOT PROVIDED

- Microscope Slides and Coverslips
- Centrifuge
- Lugol's Iodine
- Microscope

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