REAL MINI & MIDI SYSTEM TOTAL FIX

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1. PRODUCT DESCRIPTION

The Mini & Midi System Total Fix is a device designed for the collection, preservation, transport, filtration and concentration of intestinal parasites present in faeces. The system offers a fast, clean and safe protocol, avoiding direct contact with the sample and reducing the risk of contamination or exposure.

It is available in two formats:

- **Mini System:** contains 4 ml of fixative and is compatible with centrifuge rotors and adapters for 15 ml conical tubes.
- **Midi System:** contains 10 ml of fixative and is compatible with centrifuge rotors and adapters for 50 ml conical tubes.

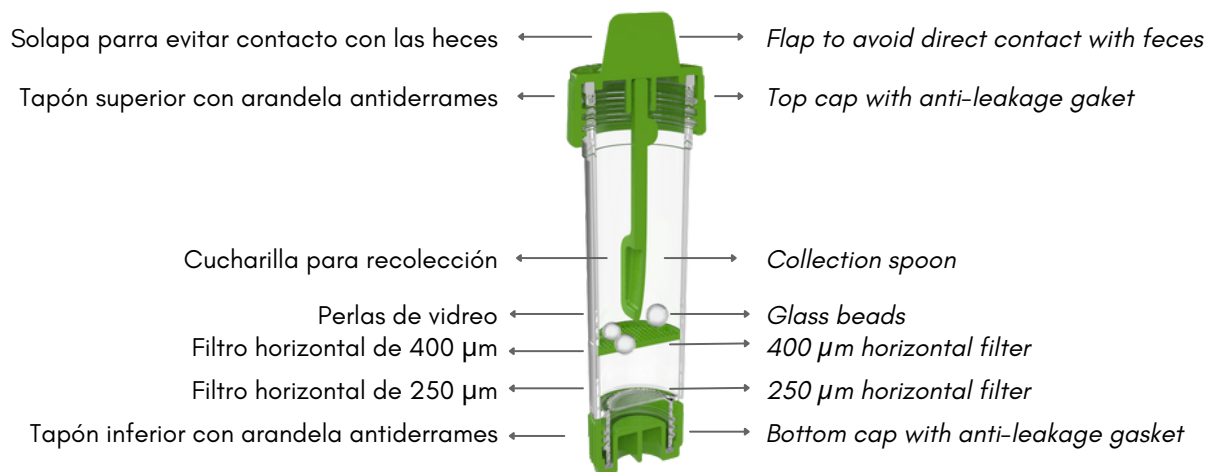
Total Fix fixative is a mixture of alcohols with zinc sulphate, formaldehyde-free, safe, efficient and compatible with molecular techniques such as PCR.

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2. TECHNICAL SPECIFICATIONS

- The tube material is polypropylene, 100% recyclable.
- Total Fix fixative volume: 4 ml (Mini) /10 ml (Midi).
- Compatible with centrifuge rotors and adapters for 15 ml (Mini) and 50 ml (Midi) conical tubes.
- Contains 3 mm diameter glass beads for easy homogenisation.
- Leak-proof seals incorporated in both caps.
- Double horizontal filter of 400 and 250 μm .

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3. STORAGE AND DISPOSAL

- Store at room temperature.
- Use before the expiry date indicated on the label. If the sample is preserved in the fixative, it will remain stable until that date.
- Dispose of according to specific methods for the destruction of material containing biological content.
- Dispose of the fixing medium through an authorised hazardous waste manager.

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4. PRECAUTIONS

- Wash hands before and after handling the sample.
- Treat samples as potentially infectious.
- Avoid using anti-diarrhoeal medication or laxatives before collection.
- **Do not ingest.** If swallowed: Do not induce vomiting. Rinse mouth with water. Seek medical attention immediately and contact your local poison control centre.
- Avoid contact with skin, eyes and mucous membranes. Do not inhale vapours. In case of contact, wash with plenty of water and seek medical attention.
- Keep out of reach of children.
- Do not expose to heat sources or direct sunlight.
- Do not pour down the drain or dispose of with municipal waste.